Safety Lancets

Lancettes de Sécurité | Lancetas de Seguridad

English

INTENDED USE: The safety lancet is intended for capillary blood sampling. Intended population: Adults and pediatrics. If a patient is under the age of 18, the lancet must be used by an adult or under the supervision of an adult. Storage Conditions: Store in a dry and well-ventilated room. Do not store virulent, harmful, or corrosive materials.

INSTRUCTIONS FOR USE:

1. Wash hands with soap and water and dry thoroughly.
2. Apply alcohol to sampling site and air dry.
3. Twist off the protective cap and discard it.
4. Put the safety lancet against the disinfected sampling site and press the lancet quickly to activate.
5. Apply gentle pressure to the skin around the sampling site to obtain blood sample.
6. Discard the used safety lancet into an appropriate sharps container.
7. Wash hands thoroughly again after blood sampling.

Note: To reduce the risk of accidental needle sticks, the needle tip will be automatically retracted once activated.


WARNINGS:

1. For single use only. Discard the safety lancet after use.
2. Do not use if the protective cap is removed or broken previously.
3. For single patient use only. Do not use on more than one patient. Improper use can increase the risk of inadvertent transmission of bloodborne pathogens, particularly in settings where multiple patients are tested.

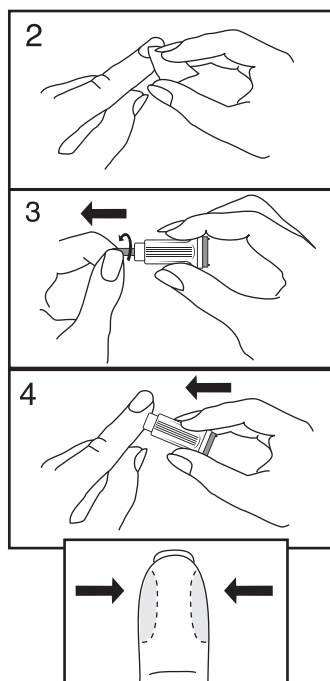Figure 2 - 4

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