
SAFETY^{1st} Pen Needles

Figure 1 Components

Figura 1 Componentes

Schéma 1 Contenu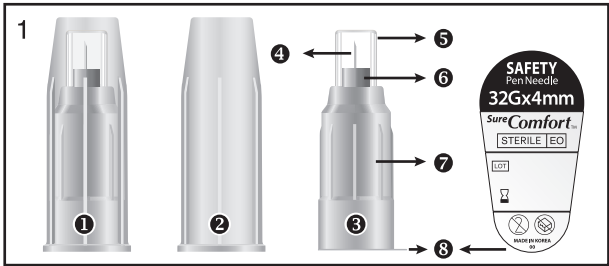

- | | | |
|-----------------|--------------------------|------------------------------|
| 1 Components | • Componentes | • Contenu |
| 2 Sterile Cap | • Tapa Estéril | • Capuchon stérile |
| 3 Pen Needle | • Pluma | • Aiguille de Stylo |
| 4 Needle | • Aguja | • Aiguille |
| 5 Needle Cap | • Tapa de la Aguja | • Couvre-aiguille |
| 6 Safety Shield | • Casquillo de Seguridad | • Gaine de protection |
| 7 Hub | • Centro | • Zone de filtage |
| 8 Sterile Paper | • Papel Estéril | • Papier d'emballage stérile |

Figure 2

Figura 2

Schéma 2

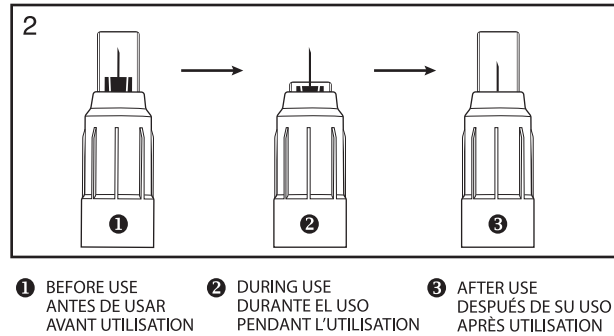
Figure 3

Figura 3

Schéma 3


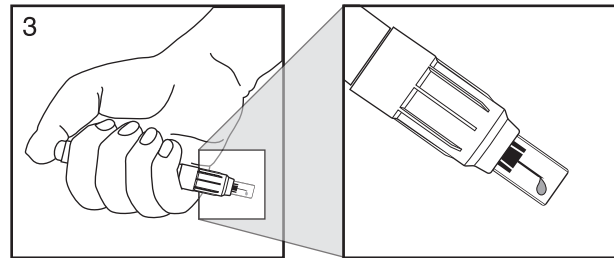
Figure 4

Figura 4

Schéma 4

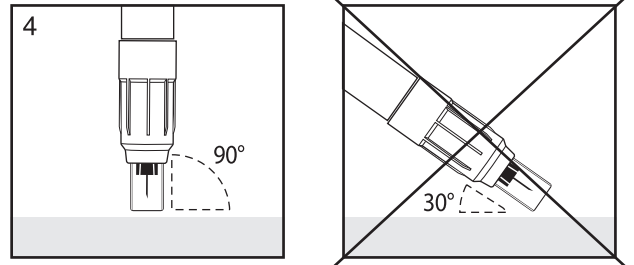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Symbol Definitions

STERILE EO	Sterile by Ethylene Oxide	REF	Reorder Number		Use By
	Do Not Use if Package is Damaged	LOT	Lot Number		Single Use
	Attention, See Instructions for Use		Manufactured By		Non Pyrogenic
	Not Made with Natural Rubber Latex	Rx ONLY	Prescription Only	DEHP Free	DEHP Free

English

INSTRUCTIONS FOR USE

INTRODUCTION

SureComfort™ Safety Pen Needle is a single-use pen needle with an automatic safety lock that is designed to reduce accidental needlestick injuries. The red indicator indicates that the needle is ready to be used. If the red indicator not visible, the locking sequence has been initiated, and the needle cannot be used. An audible and tactile “click” indicates the safety mechanism has activated. Please read and understand these instructions before using Autokeeper®

INTENDED USE

SureComfort™ Safety Pen Needle is intended for use with insulin pens for the subcutaneous injection of insulin. Additionally, the attached safety shield automatically locks following an injection to help prevent the occurrence of accidental needlesticks from the patient end of the needle.

WARNINGS

- Do not use if package is damaged.
- Single Use Only. Do not reuse or resterilize.
- Do not use the pen needle after use-by date.
- Clean the injection site per institutional protocol.
- Do not place your fingers on or near the needle cap.
- Do not bend the needle before, during or after an injection. Bending the needle may cause the needle to break while in the skin which may require medical attention to remove.
- Keep insulin needles out of the reach of children.
- Always use a new pen needle for each injection.
- Keep fingers away from the back of a used safety pen needle.
- Do not share your needles with other people. Sharing needles can lead to the transmission of diseases like HIV and Hepatitis.
- Dispose of used sharps in accordance with all local and Federal (U.S.) laws.

CAUTIONS

- If you notice a drop of liquid disappears at the needle tip, dial 2 units, press the button and check it.
- Follow the instructions of the insulin pen you are using.
- Select the dose prescribed by your healthcare professional.
- In order to avoid hardening of the skin, change the injection site regularly in consultation with your healthcare professional.
- Use ONLY compatible pens with SureComfort™ pen needles.
- The safety mechanism activates after use. Do not attempt to unlock a locked safety feature.

INSTRUCTIONS FOR USE

- Clean the rubber seal of the insulin cartridge inside the pen with an alcohol swab.
- Remove the sterile paper from the SureComfort™ pen needle.
- While holding the sterile cap, push and twist the pen needle straight onto the pen in a clockwise direction until it is fully seated.
- Pull the cap straight off.
- Check the flow of the pen needle by pressing the button on the pen. A drop of liquid should appear at the needle tip. (Figure 3)
- Dial the prescribed dosage on the pen.
- Insert the pen needle straight into the skin at a 90° angle until the needle cap retracts and the middle cap is flush with the skin. (Figure 4)
- Inject insulin. Leave the needle in the skin for 10 seconds following the injection to help ensure a complete delivery of insulin.
- The safety mechanism activates while withdrawing the needle from the skin. An audible and tactile “click” indicates the safety mechanism is activated.
- Twist the pen counterclockwise to remove the pen needle from the pen.
- Immediately dispose of the used pen needle into an approved sharps container per institutional protocols, local and Federal (U.S.) laws and regulations.

STORAGE

Store the pen needles in a dry place at room temperature (1-30°C) and keep away from hot and dry conditions.

Compatible Pens • Plumas Compatibles • Stylos Compatibles
Auto Pen / Novo Pen 4 / Novo Pen 5 / Novo Pen Echo / Humapen Luxura
Humatrophen / Humapen Savvio / Junior Star / Allstar Pro

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