Insulin Syringes

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Sterile Single-Use Syringes
For U-100 Insulin

Symbol Definitions

STERILE EO	Sterile by Ethylene Oxide
REF	Reorder Number
LOT	Lot Number
	Single Use
	Do Not Use if Package is Damaged
	Non Pyrogenic
	Not Made with Natural Rubber Latex

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INTENDED USE: For the administration of insulin. For immediate use.**INSTRUCTIONS FOR USE:**

1. To expose plunger, remove large white cap.
2. To expose needle, pull small orange cap STRAIGHT off, being careful not to bend needle.
3. To measure correct dosage, align top edge of plunger tip with your dosage mark on the syringe scale.
4. Use syringe ONCE only. To prevent misuse after injection, destroy and contain syringe and needle.

Warning: For patients who mix two types of insulin, do not change:

- A) the order of mixing that the physician has prescribed or;
- B) the model and brand of syringe without first consulting your physician or pharmacist. Failure to heed this warning can result in dosage error.

Sterility of fluid path guaranteed unless syringe is opened or damaged.**WARNINGS:**

1. Do not use if needle is bent.
2. Do not use if package is damaged.
3. Single Use Only. Do not reuse or resterilize.
4. Do not use the syringe after use-by date.
5. Clean the injection site per institutional protocol.

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