
Lancing Pen

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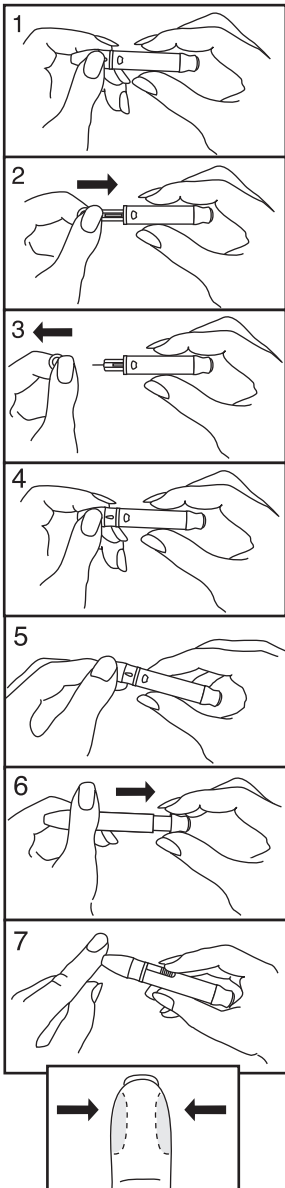Figura 1 - 7

English

INTENDED USE: This Lancing Pen is used to obtain capillary whole blood samples needed for blood glucose monitoring or for other tests requiring one or two drops of blood. The Adjustable Tip on the Lancing Pen allows you to choose the best depth of skin penetration and can be used for alternate site testing. To obtain a drop of blood, first wash your hands with warm, soapy water. Dry your hands thoroughly before using the Lancing Pen.

INSTRUCTIONS FOR USE:

1. Twist off the Lancing Pen cover.
2. Insert a new lancet firmly into the lancet carrier.
3. Hold the lancet needle cover and gently twist off the lancet protective shield.
4. Replace the Lancing Pen cover.
5. The Adjustable Tip offers 5 levels of skin penetration. To select the desired depth, twist the Adjustable Tip in either direction until the numbers line up with the arrows. To select the best depth:

1-2	For soft or thin skin
3	For average skin
4-5	For thick or calloused skin
- Note:** For finer adjustment, the arrow may be lined up between any two numbers.
6. Pull the rear of the Lancing Pen completely out until it clicks to ready the device.
7. Position the end of the Adjustable Tip against the side-tip of the finger. Press the end of the Lancing Pen until the Pen is activated. Lift the Lancing Pen away from the finger after the puncture is complete.
8. Remove the Adjustable Tip from the selected site gently.
9. Apply blood to test strip and perform test.
10. After performing your blood glucose test, unscrew the Lancet Pen Cover and push the slide on the side of the device to eject the lancet for disposal.
11. To dispose of the used lancet safely, discard it into the appropriate sharps or biohazard container. DO NOT re-use the lancet.

TAKING CARE OF YOUR LANCING PEN: Remove the Lancing Pen cover. Wash the Lancing Pen cover in warm, soapy, water. Rinse well and allow it to dry thoroughly. Use alcohol or mild soap and water to wipe the remaining outside of the Lancing Pen. The Lancing Pen can be disinfected using 70% rubbing alcohol. (Hospitals should consult their own infection control protocols.)

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